Viscoartro®

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CE 0318

(Hialuronato sódico 2% (20mg/ml) jeringa intraarticular)

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DESCRIPTION: Sodium hyaluronate is a natural polymer from the glycosaminoglycan class (acid mucopolysaccharides) widely distributed in connective tissues of animals and humans. It is a polysaccharide consisting of repeated units of the disaccharide formed by N-acetylglucosamine and sodium glucuronate bound by alternating β-1,3 and β-1,4 glycosidic bonds. It is the major component of joint fluid and plays a significant role in the physiological functions of that fluid. Viscoartro is a sterile solution with a concentration of 2% sodium hyaluronate obtained from *Streptococcus zoepidemicus* by fermentation processes and subsequent purification. It is characterized by its viscoelasticity, hygroscopicity and lubricating properties. Viscoartro is supplied in single-use prefilled syringes which facilitates administration, maintains sterility and prevents from particle contamination when in use. **INDICATIONS:** Viscoartro is indicated to reduce pain and lack of mobility due to degenerative and traumatic changes of the knee joint and other synovial joints associated with osteoarthritis. **CONTRAINDICATIONS:** The product is contraindicated in patients with a history of hypersensitivity to the product. **PRECAUTIONS:** This product may worsen local inflammation in patients with knee osteoarthritis associated with joint inflammation and effusion. Therefore, it is advisable to administer the product after the local symptom has alleviated. Local pain may rarely appear after administration of the product per se. To avoid pain after the intra-articular injection, it should be recommended to keep the joint still after the administration procedure. The product must be injected strictly into the intra-articular space, otherwise it may cause pain. Viscoartro must be administered with caution to patients with a history of hypersensitivity to other drugs and patients with liver failure or impairment. The product should be administered with caution in patients with impaired physiological functions. *Use in children:* Since the safety of this product has not been established in children, it must be administered with caution to this population. *Use in pregnancy and breast-feeding:* The safety of the product has not been established in pregnant women. Nevertheless, although no teratogenic effects have been detected in animals, it is recommended to administer the product with caution to pregnant women or women suspected of being pregnant it is recommended to discontinue breast feeding during treatment since it has been proven that sodium hyaluronate is excreted in milk of treated animals, so it is recommended to discontinue breast-feeding during treatment. **WARNINGS:** Keep out of the reach and sight of children. Do not use the product if the product package is damaged. Discard any opened or damaged container. Administer only if the solution for injection is completely clear. Keep the pre-filled syringe in the original packaging. Graduated marks on the syringe label should only be used as a guide. **DOSAGE AND ADMINISTRATION:** A single intra-articular injection of Viscoartro is recommended. In most

cases, one administration per year is sufficient. Administration should be into the intra-articular space of the knee joint and other affected synovial joints. Administration should be performed by a specialist with experience in intra-articular administration. **Cautions in use:** *Injection:* • Since Viscoartro is injected into the intra-articular space, the whole process must be performed under strict sterile conditions. • If even small retentions of synovia are noted, drain by arthrocentesis. **Others:** • Be specially careful to avoid injection of Viscoartro into blood vessels. • Do not apply the product into the eyes. • Given the viscosity of the product, the use of 21G needles is recommended. • If refrigerated (see storage conditions), allow the product to reach room temperature before use. • Use the product immediately after opening, as sterility cannot be guaranteed otherwise. The product should only be used once, and always in a single patient, and any remaining product should be discarded. Otherwise, cross infections may occur. **INCOMPATIBILITIES:** Avoid contact with disinfectants such as quaternary ammonium salts, including benzalkonium chloride, or which contain with chlorhexidine, as product precipitation may occur. **UNDESIRABLE EFFECTS:** After intra-articular administration of hyaluronic acid, some local effects have been reported, such as (transient) pain, swelling and/or joint effusion at the treated joint. Hypersensitivity reactions have also been reported, such as anaphylactic reactions, anaphylactic shock and angioedema. In these cases, the administration of the product should be discontinued, the patient must be monitored and the appropriate treatment provided. Systemic undesirable effects occasionally reported after intra-articular administration of hyaluronic acid are: rash, urticaria, pruritus, pyrexia, chills, peripheral oedema, headache, dizziness, nausea and muscle spasms (cramps). Any serious incident involving the product should be reported to the manufacturer Meiji Pharma Spain, S.A. and to the competent authority of the Member State where the user is established. **INFORMATION TO BE PROVIDED TO THE PATIENT WITH AN IMPLANTED PRODUCT:** This product, as it is considered an implant, includes an implant card that must be given to the patient once it has been filled in together with its instructions, so that he/she has the correct identification of the product that has been implanted. Due to the characteristics of the product, no warnings, precautions or special measures additional to those already indicated in the corresponding sections are considered necessary, neither on the part of the patient, nor on the part of the health professional with respect to reciprocal interferences with external influences, medical examinations or reasonably foreseeable environmental conditions. Once implanted, no monitoring of the product beyond that required by the disease itself is required. The substances to which the patient is exposed after implantation are described qualitatively and quantitatively in the Composition section. **CONTENTS OF THE PACK:** Viscoartro is supplied in single-use glass syringes, containing 4,9 ml of sodium hyaluronate (98 mg), with a Luer Lock fixing system. Syringe content is sterile, endotoxins free and non-pyrogenic. Packs containing 1 syringe. **SHELF LIFE:** Do not use the product after the expiry date which is stated on the carton. **STORAGE CONDITIONS:** Store the product between 1 and 30°C. If this temperature cannot be guaranteed, store in the refrigerator. Protect from light. **Fragile. DISPOSAL:** Place the used syringe in a sharps container immediately after use. Do not dispose of the Viscoartro prefilled syringe in household waste. Local regulations on the proper disposal of the sharps container should be followed. **MANUFACTURER:** Meiji Pharma Spain, S.A. Avda. de Madrid, 94. 28802 Alcalá de Henares, Madrid (Spain). **DISTRIBUTED BY:** See information in the carton. The Summary of Safety and Clinical Performance (SSCP) of this product is available on the European database for medical devices (Eudamed), where it is associated with its Basic UDI - DI: 8430433HA298ART96. Eudamed public website: <https://ec.europa.eu/tools/eudamed>. Revision date: 11/2023.

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HOW TO USE Viscoartro

Before use the product, if it is stored in the refrigerator (see storage conditions), take it out twenty or thirty minutes before use, and follow the instructions below.

1. Take the syringe out of the blister pack.
2. Remove the stopper by turning.
3. Insert a 21G -23G injection needle.
4. The syringe is ready for use.

*Before administration, disinfect the injection site thoroughly.
*After administration, the syringe and needle can be disposed of separately.

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